

# Fitted Face Mask

## Step by Step

Finished Mask




# Step 1

Cut 4

OR

2 fabric/2 contrast

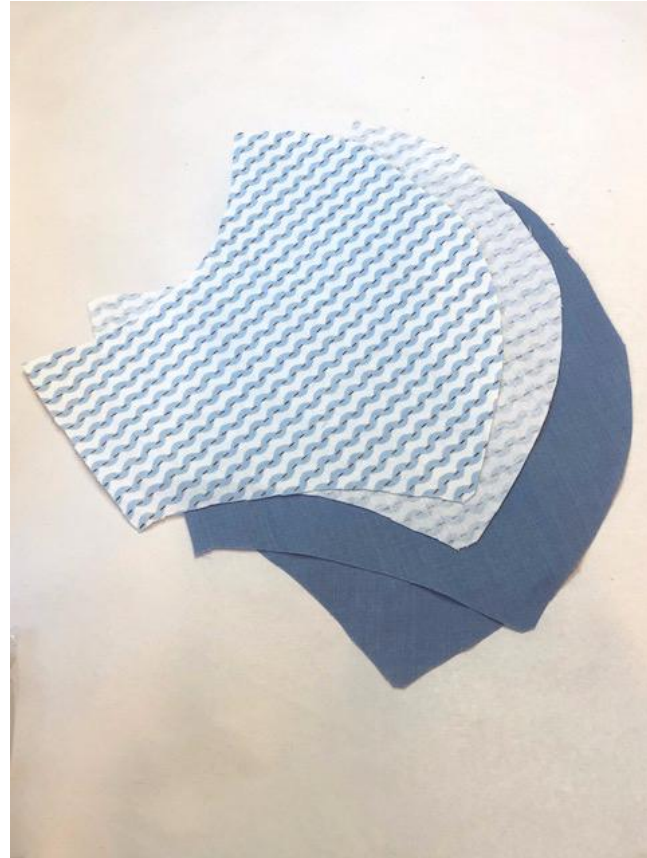

## Step 2

Sew each together  
at front curve



## Step 3

Clip curves



## Step 4: Optional

Top stitch outside  
curve



## Step 5

Turn out and press



## Step 6

Hem all 4 sides  $\frac{1}{2}$ "





## Step 7

With wrong sides together, sew around top and bottom of mask



## Step 8

Always using  $\frac{1}{4}$   
seam allowance



## Step 9

Leave sides open



## Step 10

Cut 2 ties or pre-  
made Bias Tape  
Top 40"  
Bottom 36"



## Step 11

Aligning centers, add ties to top and bottom of mask using  $\frac{1}{4}$  " seam allowance



## Step 12


If making your own Bias Tape fold in sides of ties  $\frac{1}{4}$ " on each side to center and then fold together and press.





## Step 13

Place 5-6" nose wire  
at center of top on  
seam allowance  
facing the outside

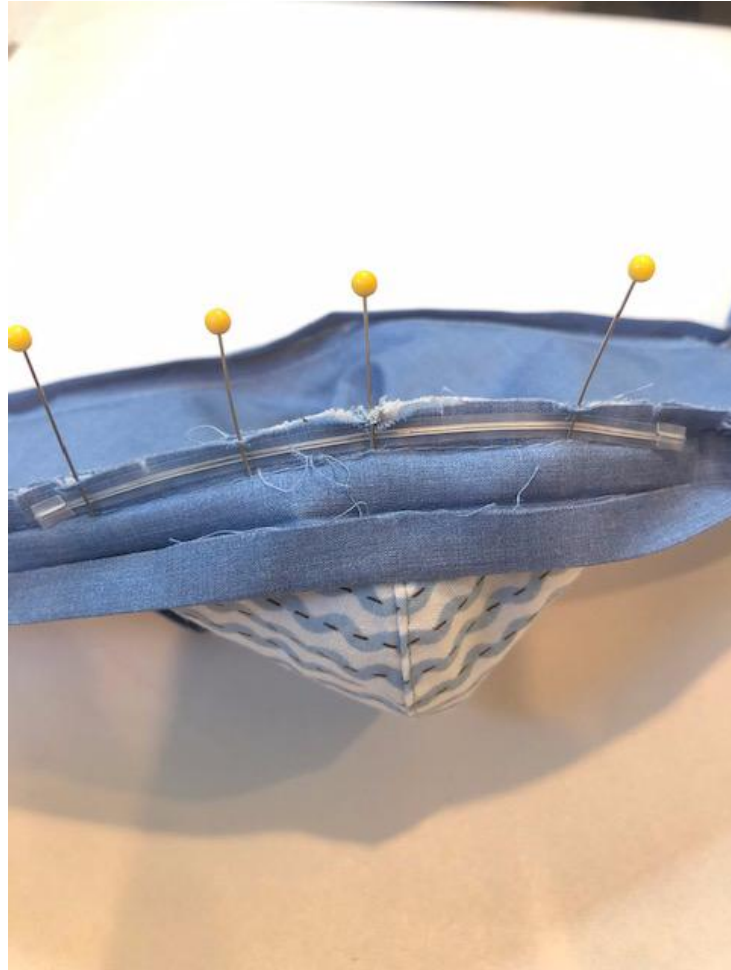

## Step 14

Zig zag nose wire in place





## Step 15

Fold ties to inside and  
sew as close to edge  
as possible and press



Finished  
Fitted mask

